Abnormal Control Plasma Art.No: GHI169

SVENSKA

Onormal kontrollplasma

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Abnormal Control Plasma

RANGE OF APPLICATION

For quality assurance or control of the coagulation analyses: PT Owren, PT Quick, Activated Partial Thromboplastin Time (APTT), Fibrinogen (Fib) and Antithrombin (AT).

For In Vitro Diagnostic use.

PRODUCT DESCRIPTION

GHI169 is a freeze-dried preparation of processed citrate-anticoagulated human plasma with additives of protein components from bovine plasma. GHI169 contains contents of coagulation analytes that are outside the reference values for normal individuals.

Package: 10 x 1 mL.

PRECAUTIONS

This product contains substance of human origin and is to be considered as potentially infectious. The product must be treated as plasma sample from a patient. For the purpose of minimizing the danger of infection, every single unit of plasma that has been used during the production has been checked and found free from antibodies against HIV, hepatitis B virus and hepatitis C virus.

ANALYTE CONTENT

For lot specific results see certificate at MediRox homepage www.medirox.se "Downloads"

Typical results for GHI169 are listed below:

PK	INR 2,3–2,9 (16-24%)
APTT	50-75 s Roughly 2-2,4 time normal time, reagent dependent
Fibrinogen	1,0-1,5g/L
AT	0,35-0,55 IU/mL

It is up to each laboratory to set their own limit values.

REKONSTITUTION

- Before the bottle is opened, carefully tap the bottle against an underlayer to let the powdered material cumulate in the bottom.
- Add 1,00 mL water of good laboratory quality. The water temperature should be 15 to 25°C.
- After addition of water, reseal the bottle and rotate it between your palms in an upright position, backwards and forwards to dissolve powder residues on the interior surface of the bottle.
- After approx. 15 minutes the control plasma is ready for use. Control visually that all material is completely dissolved, if not use a mixer.

STABILITY AND STORAGE

Unopened reagent is stable until the expiration date shown on the label when stored at 2-8°C.

Stability after reconstitution:

24 hours at 15-25°C in the closed original vial,

24 hours at 2-8°C in the closed original vial. (Make sure the control reaches room temperature before use).

Revision: 2016-04-11

Changes: New logotype and text format

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